

Access Your CGM Data

Your Eversense Data Management System (DMS) account is used by your Eversense mobile app, and by the Eversense DMS web app for data review and reports. **If you have already downloaded the Eversense App and activated it through the email, please skip to step #3.**


1 Creating your Eversense account

- Download and install the Eversense Mobile App


Download the free *Eversense App* from the Apple App Store or on Google Play*.



- After you accept the License Agreement, you will be prompted to create and register an account with an Email and Password.
- After completing the screen, tap the arrow on the bottom of each page to advance to the next screen.

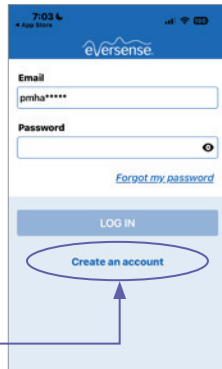


Tap **Create an account**.

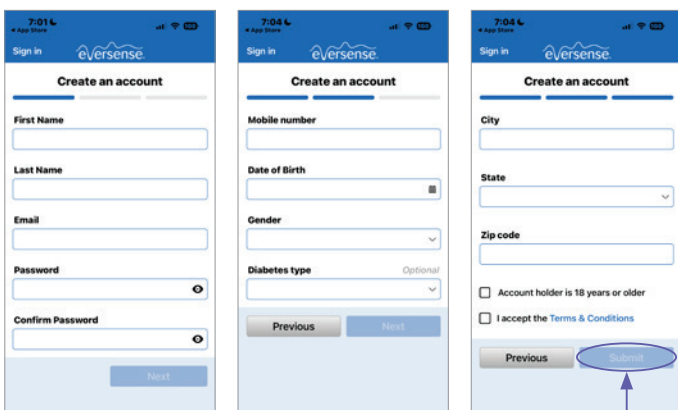

Click **Submit**.

2 Activate your Eversense account

Once you have submitted your account information through the app, you will receive an email from Eversense with a link to complete the registration and activate your account. **You may seamlessly complete this process directly through your mobile device.**

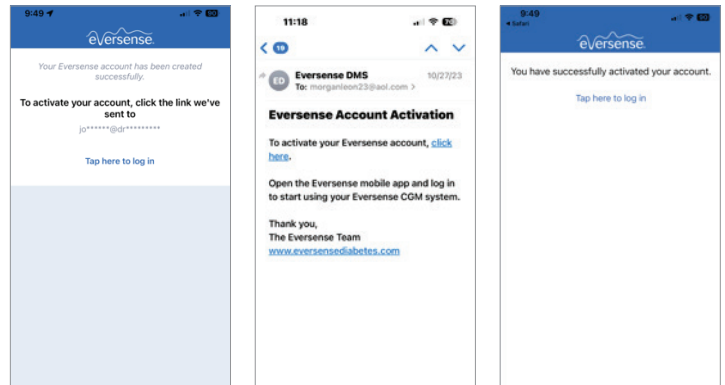

Important: Within 72 hours, click on the link in the email to activate your account.

Congratulations! Your account is now activated. You can click on the link to log in and see your glucose data reports.

3 For future reference, record your DMS account log-in information below

Email: _____

Password: _____

4 Accessing your DMS account via your computer

You are always logged into your account through the Eversense App, but to get full access to all your data just go to: <https://us.eversensedms.com/> and enter your log-in information.

Remember your log-in information is the same as what you used when you created your Eversense account.



5 Allowing Your Health Care Provider Access to Your Data


Clinic Share is an optional feature on your Eversense Mobile App that allows you to either invite or accept an invitation from your health care provider's clinic via Eversense DMS Pro. This will allow them to review your Eversense DMS glucose reports.[†]

To invite your health care provider:

You will need their Eversense DMS Pro **Clinic ID** number, which they will need to provide to you. Please contact them directly.


1. Tap **Menu > Share My Data > Clinic Share**
2. Tap **Clinic Requests > Send Requests**
3. Enter your health care provider's Eversense DMS Pro **Clinic ID** number
4. Tap **Add**
5. Tap **OK** to confirm invitation

Your provider will now be able to view your data from their Eversense DMS Pro account.

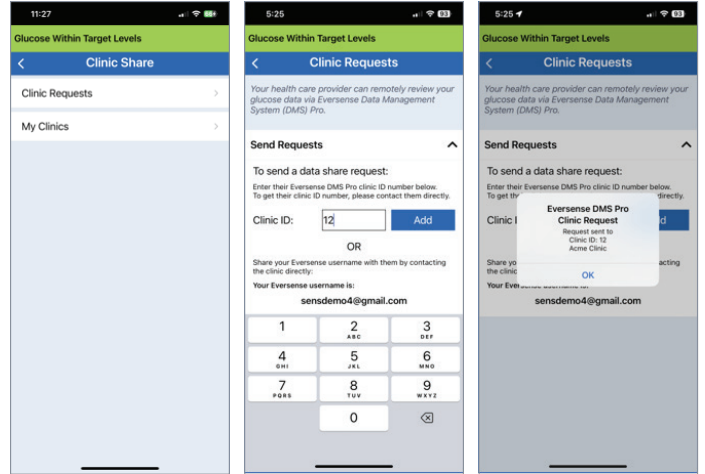

6 Viewing, Printing and Emailing Reports

1. Go to: [https:// us.eversensedms.com](https://us.eversensedms.com) and enter your log-in information.
2. On the left navigation pane, click on **Reports** to open the list of reports.

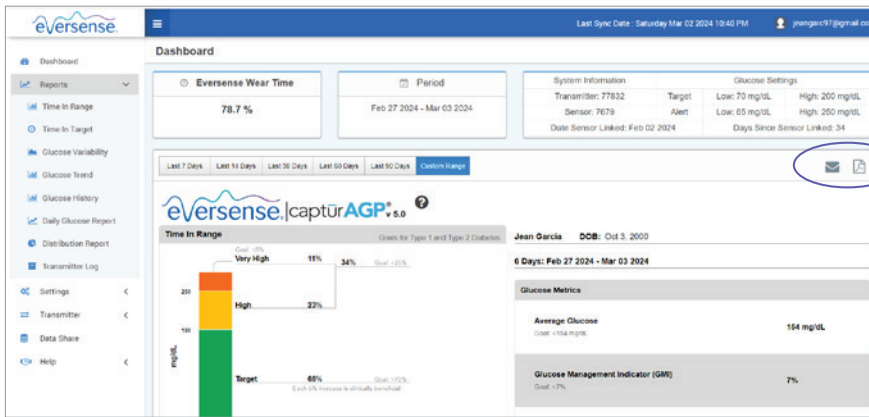

You may print or email your reports. Click on the printer icon to view and print your report or click on the envelope icon to email them.

Located in the settings menu, **REPORT SUMMARY** settings allow you to set up a regularly scheduled automated report that will be emailed to you. You can select how often the reports are sent, and which reports you want to receive.

7 Understanding Your Eversense DMS Reports

Learn about how your data can help you manage your diabetes by going to www.eversenseddiabetes.com. Click on **Patient Education**.

To contact Customer Care, call 844-SENSE4U (736-7348)

[†] The app applies appropriate safeguards to ensure your personal data is processed securely and in compliance with applicable laws.