

USING YOUR IN-APP REPORTS

Your Eversense CGM System automatically creates 3 reports which you can view at any time right in your app. These reports may help you and your health care provider fine tune your diabetes management plan.

➤ To access your In-App Reports, go to **Main Menu > Reports**

The Weekly Modal Summary report:



- Provides a summary of your last seven days of glucose information in a 24 hour line graph format
- Makes it easy to see glucose trends by time of day
- **Red bars** | show amount of glucose variability at a given time
- Scroll down to see additional glucose statistics based on pre-set time slots

The Glucose Pie Chart report:

- Shows time spent within your target range, below target range, and above target range
- Is viewable for 1, 7, 14, 30, or 90 days of data
- Comparing data at different time intervals may help you and your health care provider see how changes to your diabetes care plan are working

The Glucose Statistics report:

- Provides average, low, and high glucose values based on preset time slots
- Is viewable for 1, 7, 14, 30, or 90 days of data
- Comparing data at different time intervals may help you and your health care provider see how changes to your diabetes care plan are working

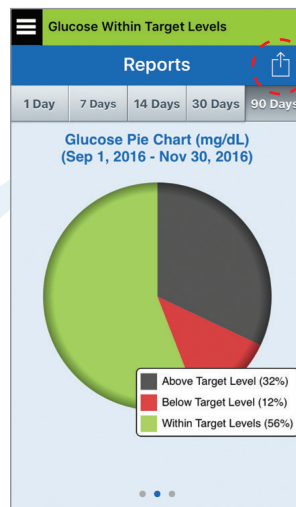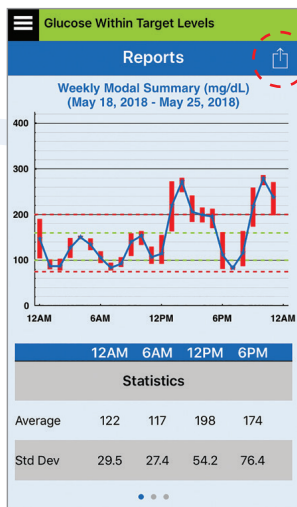

Glucose Statistics (mg/dL)
(May 24, 2018 - May 25, 2018)

Period	Avg.	Low	High	Std Dev
12AM - 6AM	122	78	190	29.5
6AM - 12PM	113	78	163	26.7
12PM - 6PM	197	93	280	50.0
6PM - 12AM	174	80	286	76.4
ALL	150	78	286	60.6



All reports can be viewed, emailed or printed directly from the app!

The Eversense® E3 Continuous Glucose Monitoring (CGM) System is indicated for continually measuring glucose levels for up to 180 days in persons with diabetes age 18 and older. The system is indicated for use to replace fingerstick blood glucose (BG) measurements for diabetes treatment decisions. Fingerstick BG measurements are still required for calibration primarily one time a day after day 21, and when symptoms do not match CGM information or when taking medications of the tetracycline class. The sensor insertion and removal procedures are performed by a health care provider. The Eversense E3 CGM System is a prescription device; patients should talk to their health care provider to learn more.

For safety information, see <https://www.ascensidiabetes.com/eversense/safety-info>

Eversense, Eversense E3 Continuous Glucose Monitoring and the Eversense logo are trademarks of Senseonics, Incorporated. Ascensia, the Ascensia Diabetes Care logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.

© 2022. Senseonics, Inc. All rights reserved.
© 2022. Ascensia Diabetes Care Holdings AG. All rights reserved.



Distributed by Ascensia Diabetes Care
5 Wood Hollow Road, Parsippany, NJ 07054
844-SENSE4U (736-7348)
ascensidiabetes.com/eversense



Continuous Glucose Monitoring System
Manufactured by Senseonics, Inc.
20451 Seneca Meadows Parkway
Germantown, MD 20876-7005